




venome®

MICROGRAFT
TECHNOLOGY IS NOW
**MORE EFFECTIVE
& FASTER**

Microlyzer Blade Technology
High Cell Count and Viability
No Device Required System



venome®

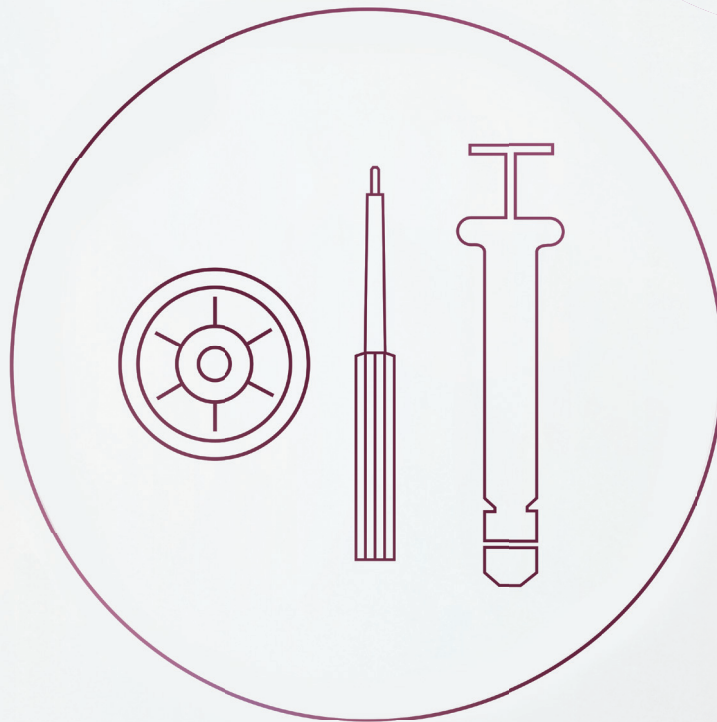

TISSUE GRAFTING KIT

Why Venome Micrograft Technology?

Without the need for any device, a system that you can perform only by using the Tissue Grafting Kit.

Providing ease of use with Microlyzer 600µm, which has thousands of blades inside, micrograft technology where you can achieve high cell count and viability.



Microlyzer 600µm

Through thousands of specially designed blades in it, the tissue pieces processed from the blades and freeing the cells around its micro environment, thus maximizing the number of cells and their vitality.




Microlyzer 150µm

With the released cell suspension, it breaks down unwanted fibrotic tissues and becomes an injectable form, to prevent this situation and ensure safe injection, the tissue suspension is passed through Microlyzer 150µm once and the unwanted remnants are filtered out.



How to use?

Every materials you need, from the sterile tablecloth to the injection needle, are in your Tissue Grafting Kit.

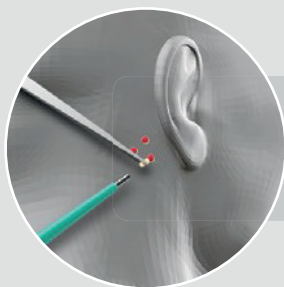

Grafts are taken from the sterilized and local anesthetized area.



By choice, it can be mixed with Saline and/or PRP and passed from Microlyzer 600µm 51 times.



Pass 1 time through Microlyzer 150µm to filter out Fibrotic and unwanted tissues (if any).



General Indications

- Facial Rejuvenation and Renewal
- Removal of Wrinkles and Sagginess
- Increase Skin Brightness
- Post-Hair Transplantation and Hair Regrowth
- Burn Treatments
- Treatment of Surgical Scars